

FIRST CHECK™



IMPORTANT NOTE:

When reading your results, a line appearing next to the word Drug represents a **Negative Result**. **NO** line appearing next to the word Drug represents a **Preliminary Positive Result**.

First Check™ offers a variety of solutions for fast, accurate and reliable drug testing in the privacy of your own home. Depending on the test you have purchased, First Check can detect the possible use of up to 7 illicit and 5 prescription drugs that are most commonly abused.

Before You Test

- Read these instructions completely.
- Have a watch, clock or timer available.
- Check expiration date on the box. Do not use if product has expired.
- In order to test in your home, you will only need these items:
 - *Test Lid (in foil pouch)*
 - *Collection Cup*
 - *Insert*
- Set aside these items, they will only be needed if you choose to ship the sample for confidential confirmatory lab testing:
 - *Pre-addressed Mailing Box*
 - *Plastic Transportation Bag*
 - *Identification Label*

Step 1 Take the Test

- When you are ready to test, remove the Test Lid from the sealed foil pouch.



- Remove plastic wrapper from the Collection Cup. Fill the Cup with urine to at least the minimum fill level line.

Min. Fill Line

Step 2 Activate Test

- Twist the Test Lid securely on to the Collection Cup. Ensure the lid is fastened tightly.



- Activate the test by tipping the Collection Cup on its side, resting the Test Legs on a flat surface. The First Check logo should be facing up as shown.


Test Legs

Step 3 Read Your Results



- You can read your results after 5 minutes, but do not wait longer than 10 minutes.
- A red or pink colored line must appear next to the word Control to indicate the test has worked correctly.

First Check Lid

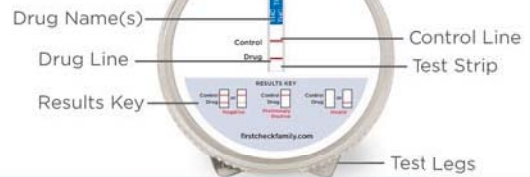

• Negative Result

A red or pink colored line next to the word Control **AND** a red or pink colored line next to the word Drug indicates a **Negative Result** for that drug, regardless of how dark or faint the lines may appear.



STRONG NEGATIVE



WEAK NEGATIVE

No further testing is required.

• Preliminary Positive Result

A red or pink colored line next to the word Control and **NO** line next to the word Drug indicates a **Preliminary Positive Result** for that drug.




• Invalid Result

If after 10 minutes **NO** line appears next to the word Control, the result is **Invalid**.



OR


- If you have any questions regarding your results, call our toll-free Help Line at **1-888-788-5716**.

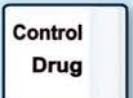


Understanding Your Test

Standard Drug Tests: Illicit Drugs Only


If you purchased one of the following tests, your Test Lid will have 1 Drug Name identified on each Test Strip.

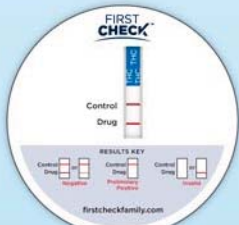


1 Drug Tested:
Marijuana (THC)

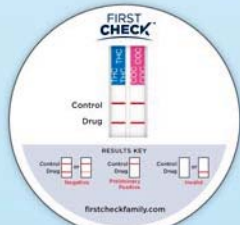


2 Drugs Tested:
Marijuana (THC) and Cocaine (COC)

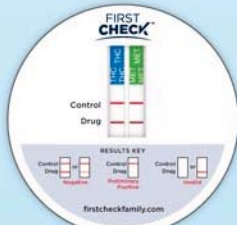

2 Drugs Tested:
Marijuana (THC) and Methamphetamine (MET)

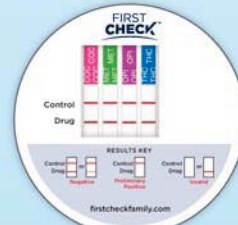


4 Drugs Tested:
Marijuana (THC), Cocaine (COC), Opiates (heroin) (OPI) and Methamphetamine (MET)

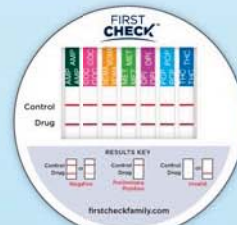

7 Drugs Tested:
Marijuana (THC), Cocaine (COC), Opiates (heroin) (OPI), Methamphetamine (MET), Ecstasy (MDMA), Amphetamines (AMP) and Phencyclidine (PCP)

Drug Test Results Area

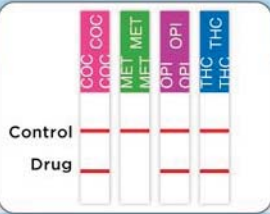


- This example shows a **Preliminary Positive Result** for Methamphetamine (MET) because there is **NO** line next to the word Drug on the second Test Strip.
- This example also shows a **Negative Result** for all other drugs.

Complete Drug Test: Illicit and Prescription Drugs

The complete First Check Home Drug Test kit tests for 12 drugs, 7 illicit and 5 prescription.

The Test Lid has 6 Test Strips in total. Each has 2 Drug Names per Test Strip. They will be identified as Drug 1 and Drug 2.

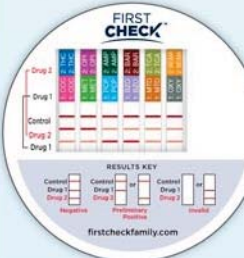


12 Drugs Tested:

7 Illicit:
Marijuana (THC)
Cocaine (COC)
Opiates (heroin) (OPI)
Methamphetamine (MET)
Ecstasy (MDMA)
Amphetamines (AMP)
Phencyclidine (PCP)

5 Prescription:
Tricyclic Antidepressants (TCA)
Barbiturates (BAR)
Benzodiazepines (BZO)
Methadone (MTD)
Oxycodone (OXY)



Drug 2 row tests for:
THC, OPI, AMP, BAR, TCA, MDMA

Drug 1 row tests for:
COC, MET, PCP, BZO, MTD, OXY

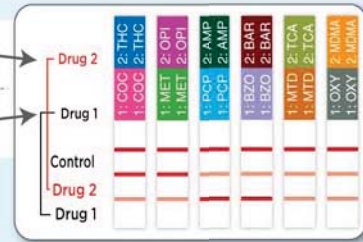

Drug Test Results Area

- If you have 3 lines in a Test Strip column, you are **NEGATIVE** for both Drug 1 and Drug 2 on that Test Strip.
- If there is **NO** line in the middle row, that indicates a **Preliminary Positive Result** for Drug 2 on that Test Strip. Match Drug 2 to the Drug Name indicated in that Test Strip.
- If there is **NO** line on the bottom row, that indicates a **Preliminary Positive Result** for Drug 1 on that Test Strip. Match the Drug 1 line to the Drug Name indicated in that Test Strip.

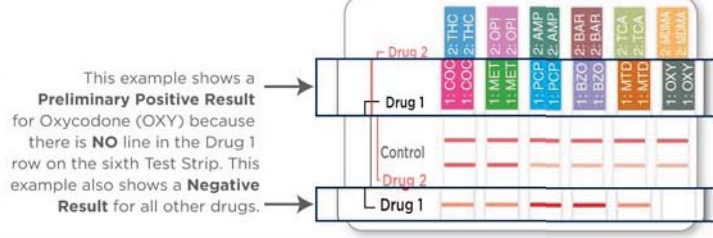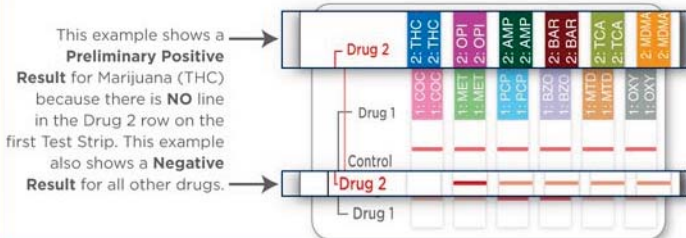

Questions?

Call our toll-free Help Line at 1-888-788-5716.

Monday - Friday
8:30 a.m. - 5:00 p.m.
Eastern Time.

Our professional staff is available to assist you.

Visit us online at www.firstcheckfamily.com

Drug Abbreviations

Illicit Drugs

Identifier	Drug
THC	Marijuana
COC	Cocaine
MET	Methamphetamine
AMP	Amphetamines
MDMA	Ecstasy
OPI	Opiates (heroin)
PCP	Phencyclidine

Prescription Drugs

Identifier	Drug
BZO	Benzodiazepines
MTD	Methadone
BAR	Barbiturates
TCA	Tricyclic Antidepressants
OXY	Oxycodone